

Adding a Quality Control Lot for Use on the IDEXX **ProCyte Dx*** Hematology Analyzer (For Research Applications without an **IDEXX VetLab*** Station)

To add a quality control lot:

1. Ensure that you have downloaded and stored the most current QVM files on a virus-free flash drive.
2. Insert the flash drive into the IPU and then open it with Windows* Explorer.
3. Create a folder on the IPU desktop called **QVM** (if one does not already exist).
4. Copy the two files from the flash drive into the QVM folder and remove the flash drive from the IPU.
5. Double-click the **QC Files** icon on the Main menu.
6. Select a QC file that has no information in it.
7. Click the **Input** button.
8. Click **Read File**.
9. Click **Browse**, select the QVM folder on your desktop, and click **OK**.
10. Select one of the files and click **OK**.
11. Click **OK**.
12. Repeat steps 6–11 for the second QC file.

IDEXX Technical Support

U.S./Canada/Latin America 1-800-544-5205

Europe 00800 1234 3399

Australia 1300 44 33 99

New Zealand 0800-102-084

Asia 0800-291-018